AIR+

SMART MASK

AIR⁺ SMART MASK

Instruction for use

1. The instructions for use and warning are guides to the proper use of the mask and to prevent health risk. They must be carefully read, completely understood and properly followed. If using the mask in work environment: the wearer of the mask must be well trained to use it properly before starting work.
2. Before using, check the mask for holes, tears or other damage to its surface, strap and/or nose cushioning to ensure that your mask works well. Replace with a new mask, if damaged or damp.
3. Select the right mask size to fit your face. Consult the given size chart on the left for size selection. If your selected size fails to fit your face properly, choose another size until the proper size to fit your face is obtained. Please refer to procedures 4 to 6 below.
4. Fold the nose clip into a  shape for a proper fit.
5. The mask strap can be adjusted for proper fit. Pull the strap outwards at either side of the buckle for tighter fit, push the strap inwards at either side of the buckle for looser fit.
6. To ensure a tight seal on your face, cover the middle surface of the mask with one hand and exhale deeply. Do not push the mask against your face. If you feel air blowing at your face or eyes, the mask's position and strap needs to be re-adjusted. Repeat the above steps until you do not feel air blowing at your face or eyes.
7. The mask should be used with the valve cap, unless it is used with the Micro Ventilator device (available separately).
8. The mask must be stored in its original packaging and stored in a clean, cool and dry atmosphere to ensure that it is protected against dust, chemicals, moisture, excessive heat and direct sunlight.
9. No alteration or modification is allowed to the mask. The mask should not be worn when it is contaminated (eg. dusty, wet, etc) or damaged (eg. broken part, hole, etc) The contaminated or damaged mask must not be used and must be replaced with a new one.
10. The mask must not be used continuously for more than 8 hours (a single work shift). Always replace with a new mask, whenever the breathing resistance becomes excessive or after using continuously for 8 hours.
11. FFP1 disposable respirators will protect against solid aerosol with concentration of toxic substance up to 4 times; FFP2 up to 10 times; FFP3 up to 20 times the threshold limit value (TLV), but it does not protect you against solvents for spray painting operation, carcinogenic and radionuclide substance, micro-organism and substance with biochemical potential, gas and vapor with harmful effect. This face mask can help protect your lungs against certain airborne contaminants, however, it will not prevent entry through other routes such as the skin, which would require additional personal protective equipment.
12. The valve cap is reusable. To clean and disinfect, please wipe its surface with a damp and soft cloth or wet tissue or alcohol wipe. Do not use strong chemical or abrasive solvent to clean the valve cap.
13. While in use, avoid touching the mask. It is important to consider the mask as potentially contaminated upon removal and user should wash hands thoroughly with soap and water or using an alcohol-based hand-rub.
14. To remove the mask, handle only the elastic bands starting with the bottom and then the top. Lift the mask away from the face and discard properly in a designated waste receptacle.
15. The mask is certified in conformity with the European Standard EN 149 and the essential requirement of European Directive 89/686/EEC, issued by the notified body 0194-INSPEC International Ltd. 56, Leslie Hough Way, Salford, Great Manchester M6 6AJ, United Kingdom.
16. The "Lot" code represents the production date in yyyy/mm/dd format. The "Exp" represents the expiry date in yyyy/mm/dd format.

L
SIZE

M
SIZE

FIND
YOUR RIGHT
MASK SIZE


Use this chart to
measure from
your chin to eye
level for the
closest size.

Safety and Precaution


1. Do not use the mask as protection against toxic particle, gas, vapor or solvent from spray-painting operation. Those who have difficulty breathing must take caution when using the mask to ensure that their ability to breathe is not further hindered by wearing the mask.
2. Do not use the mask in enclosed space, tank, inadequately ventilated area, oxygen-deficient environment (<17% in volume), hazardous or explosive environment, or in any circumstance where toxic gas is likely to be present.
3. Do not use the mask when the concentration of contaminant is dangerous for life or health. The concentration toxicological characteristics and type of substance should be known.
4. Leave the work area immediately if breathing becomes difficult, dizziness or any other sign of distress occurs.
5. Ensure proper fit on your face before using the mask.
6. The mask should not be worn under any conditions preventing a proper fit on your face. Such conditions may include growth of beard, mustache, sideburn, the use of spectacles or any other device containing straps which passes between the mask's sealing edge and the user's face.
7. If allergy occurs when mask is in contact with your skin, the mask should be removed immediately.

Fitting Instructions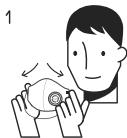

Open & Fit

With the valve on the left open, place the mask over face to check for the right fit.

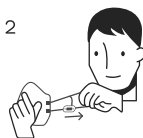Adjust Lower Strap

Pull the lower strap and fit it over your head, below your ear.

Fit Upper Strap

Stretch the upper strap over your head, above the ear.

Adjust

Press on nose clip with both hands to ensure tight fit around face.

Blow To Check

If air leaks from edges, readjust mask and straps to correct fit

Attaching the AIR+ Micro Ventilator

Twist the valve cap to the left to remove. Then attach the AIR+ Micro Ventilator

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